

ELITE Scissor lift consult & transport table - N° TA100

The consult & transport table, entirely in stainless steel, hygienic and durable material, is proving to be a real asset in any clinic situation. Very versatile, it is three tables in one : an examination table, a lift table and a transport gurney.

It protects the animal as well as the staff, as it is perfect for lifting and transporting even the largest patients, up to 100 KG, without unnecessary handling or movement, thanks to its variable height which is adaptable to the different animal's transfers throughout its medical follow-up.

Fitted with swivelling castors, movements become easier. Each castor with braking system allows to keep a fixed position at the needed location and this protects the table from any untimely movement.

Dimensions :

- Table top : 129 x 60 cm
- Minimal height : 22.5 cm
- Maximal height : 96.5 cm
- Maximum load : 100 KG

Custom sizes can be made to order


Options :

TA 100400 - Consult & transport table with battery

TA 100405 - Consult & transport table with remote control* and battery

TA 100410 - X-ray** consult & transport table with battery

TA 100415 - X-ray** consult & transport table with remote control* and battery

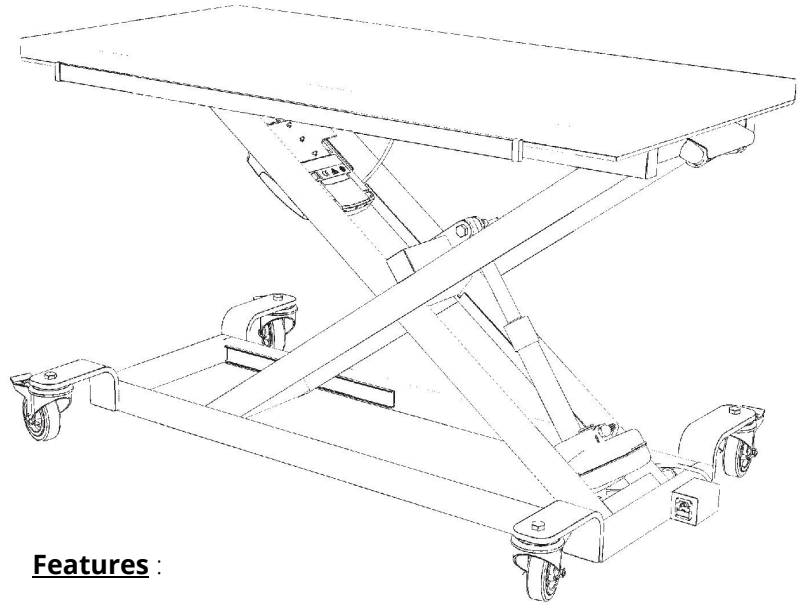

Features :

- Hand switch for height adjustment
- Electric actuator for smooth and silent up-and-down operations
- Stainless steel top with height adjustment from 22.5 to 96.5 cm
- X-frame design for maximum stability
- Entirely in 304 L stainless steel
- 360° movable castors with locking system - non marking in natural rubber - diameter of 125 mm standardized
- Tie-down bar on each side
- Built-in 24V rechargeable battery - long autonomy - 204 V charger
- All electrics are certified for EU standards
- 5 years warranty for the frame
- 2 years warranty for electric parts
- *For remote control option - wired system
- **For X-ray option - specific machined polyethylene scratch resistant coating - 16 mm - blue color



ZAC du Pont des Nautes - 03410 SAINT VICTOR

Tél : 04 70 03 31 31 - Fax : 04 70 03 03 07

E-mail : contact@vetinox.fr

www.vetinox.fr



ELITE Scissor lift consult & transport table - N° TA100

Please read carefully the user manual before operating the device

First of all, don't forget to activate your warranty directly on our website www.vetinox.fr - tab Product Warranty.
Then, let yourself be guided to register your device.

• **Set up - Activation**

- Carefully unpack the device
- Remove the foam wedges
- Connect the table to a power socket protected by a residual-current circuit breaker
- *Be careful, before the first use, let the table charge for 24 H.*
- To lift or lower the table, simply activate the hand switch.



• **Maintenance - in case of failure**

- A warranty of 5 years is offered to cover mechanical defects or corrosion as well as a warranty of 2 years for electric parts.
- This is a geared motor actuator - power 220 Volts – 400 Watts.
- Current-power = 220-240 volts + battery 24V

Your table does not rise or lower anymore :

- Check that the battery is fully and correctly charged
- Check your circuit breaker on your power switchboard
- If in spite of all these checks, your table will not operate, please contact us, we will be pleased to take care of your problem.

• **Contraindications**

- Do not immerse the table or wash it with a high pressure spray – shock hazards
- Do not exceed the maximum authorized weight
- Maximum allowable weight : 100 KG
- Connect the device to a secure electrical network
- Do not change the existing frame or the electrical system and in general do not make any modification
- Install the device in a suitable clean place

• **Cleaning and precautions for use**

The stainless steel used is type 304 L. This stainless steel has a very high resistance to corrosion. As stainless steel is a solid material and not a surface coating, the appearance of stains or traces may not be linked to rust in any case.

You can clean and restore the brushed appearance with a synthetic scouring pad, like green Scotch-Brit, by scrubbing lightly with the grain. Do not use steel wool. Ban all products in cream formulation which might clog or bleach and chlorine - based cleaners. It is better to favor glass products or quaternary ammonium.

If you have any doubt, please send us a photo of the cleaning product composition to confirm use.



ZAC du Pont des Nautes - 03410 SAINT VICTOR

Tél : 04 70 03 31 31 - Fax : 04 70 03 03 07

E-mail : contact@vetinox.fr

www.vetinox.fr

